# steptics<sup>®</sup>

sports

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## Table of contents

1. Description.....	11
2. Intended use.....	11
3. Included components.....	11
4. Indications for use and target and patient group.....	12
5. General safety instructions.....	12
6. Warnings  .....	12
8. Assembly instructions.....	13
9. Alignment instructions.....	14
10. Use.....	14
11. Maintenance.....	15
12. Report on a serious incident.....	15
13. Disposal.....	15
14. Liability.....	15
15. Conformity.....	15
16. Name and address of the manufacturer.....	15
17. Technical data.....	16

Medical device

## steptics sports Instructions for Use

Version 1.1  
Status: 26.01.26

### Information:

- Please read this document carefully before using the product.
- Follow the instructions for fitting and prosthesis assembly carefully.
- Please keep this document in a safe place.
- Instruct the patient in the safe use of the product.
- For users with visual impairments, this document is available as a PDF file for enlarged viewing on our website.

## 1. Description


This product is a prosthetic foot intended for use in a sports prosthesis. The blade and sole are connected by adhesive.

## 2. Intended use

As part of the sports prosthesis, the prosthetic foot replaces the functions of the missing ankle and foot in the lower limb and is intended for transfemoral and transtibial amputees. The suitability of the prosthetic foot for the patient and sports prosthesis as well as the fitting and adaptation of the product must be selected and fitted by a qualified orthopaedic professional.

This product is a running blade with which long jumps, jumps from great heights or activities with similarly high loads are not permitted.

## 3. Included components

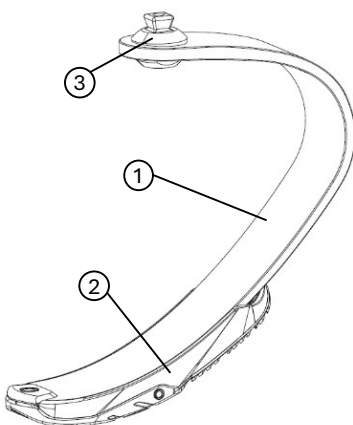No.	Designation
1	Blade
2	Sole
3	Pyramid adapter

Figure 1: steptics sports

## 4. Indications for use and target and patient group

- Loss of lower limb, amputation or deformity
- Body weight up to a maximum of 120 kg
- No known contraindications

## 5. General safety instructions

- The use of a lower limb prosthesis entails a fundamental risk of falling, which results in an increased risk of injury.
- This product is intended exclusively for the use of a single person. Reuse by other persons is not permitted.
- Assembly and adjustment may only be carried out by a qualified orthopaedic technician.
- Only use the product for its intended purpose.
- Handle the product carefully to avoid mechanical damage.
- Avoid exposing the product to unauthorised ambient conditions.
- Check the product for functionality and damage before and after each use and after contact with unauthorised ambient conditions.
- If the functionality of the product changes, its function is restricted or there are signs of damage or wear that prevent safe use, the product must no longer be used. In such a case, an orthopaedic specialist must be contacted immediately.
- All aspects listed in the instructions for use that contribute to the proper and safe use of the product should be explained to the patient by an orthopaedic technician before use.

## 6. Warnings

- This product is not designed for activities that put excessive strain on the product, such as jumping from great heights.
- At the end of its service life, the product must no longer be used, as safety and function are no longer guaranteed.
- This product is not intended as a replacement for an everyday prosthesis.
- Do not drill, grind or otherwise modify the product to ensure safe use by the patient. (Exception: roughening of the adhesive surface for fitting the outsole)
- The weight specified for the configuration of the prosthetic foot must not exceed the maximum weight specified on the product label. Using the prosthetic foot in excess of this weight limit will void the warranty and release the manufacturer from any liability. This is to ensure that the product is not subjected to loads that exceed the tested requirements and that the highest possible level of safety is guaranteed when using the product.

## 7. Product selection

Ensure that the configured variant of the product is suitable for the patient's weight limit and experience level.

User weight	Experience level	
	Low	High
40 - 50 kg	Cat. 1	Cat. 2
51 - 60 kg	Cat. 2	Cat. 3
61 - 70 kg	Cat. 3	Cat. 4
71 - 80 kg	Cat. 4	Cat. 5
81 - 90 kg	Cat. 5	Cat. 6
91 - 100 kg	Cat. 6	Cat. 7
101 - 110 kg	Cat. 7	Cat. 8
111 - 120 kg	Cat. 8	N/A

## 8. Assembly instructions

**Caution:** Sharp edges or splinters can cause cuts.

The product may only be used in combination with a Steptics insole. The outsole may only be replaced by a qualified orthopaedic technician.

**Technical data:**

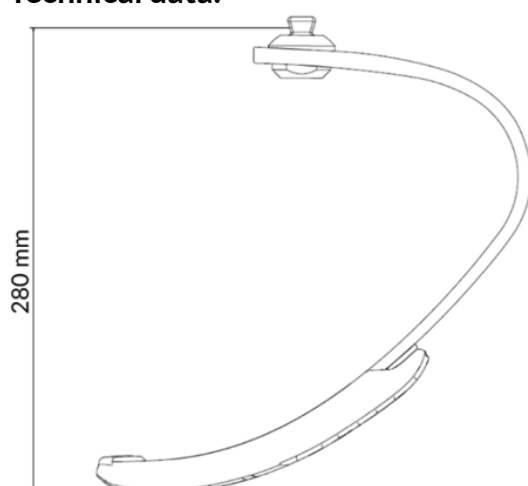
Figure 2: Installation height

## 9. Alignment instructions

### Static

#### Sagittal:

1. Place the foot module with its floor contact point in front of the reference line according to figure 3 and a dimension L of approx. 50-70 mm.
2. Set the appropriate socket flexion.

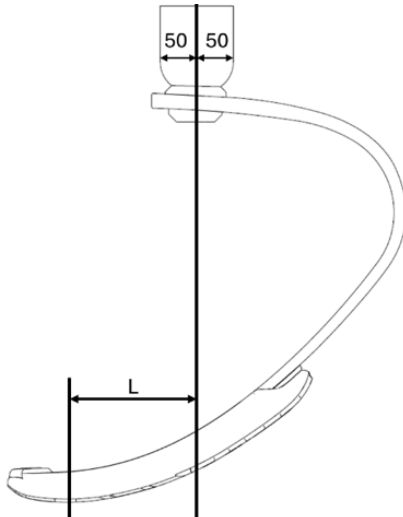
Figure 3: Dimension L, static structure

#### Frontal:

1. Note the shaft abduction/adduction.
2. Set the external rotation of the foot module between 0° and 5°.
3. Allow for a compression of approx. 30 mm in the height structure.

## 10. Use

### Cleaning and care

Rinse with clear fresh water or a damp cloth and clean with a mild soap. Then dry with a soft cloth.

### Environmental conditions

- The product is waterproof and can therefore be used in a wet or damp environment and immersed in water up to 3 metres deep for a maximum of 1 hour.
- It tolerates contact with fresh water, salt water, chlorinated water and mild soaps.
- Urine and sweat are considered inadmissible environmental conditions and may cause premature wear.
- Clean the product after contact with fresh water, moisture and other liquids, chemicals and after soiling in accordance with the care and cleaning instructions.
- Keep the product away from open flames.
- Temperature range: -10 to +60 °C
- Humidity range: 0 % to 90 %

## Service life

**Blade incl. pyramid adapter:** This product has been tested in accordance with ISO 22675 with two million load cycles. Depending on the patient's activity and frequency and intensity of use, this may correspond to a service life of 1-2 years.

**Sole:** The outsole is a wear part and is subject to normal abrasion. Its durability depends on the individual use and the respective operating conditions. Regular inspection for wear is recommended to ensure safe use.

## 11. Maintenance

The product must be checked regularly by a qualified orthopaedic technician for external damage and functional limitations. The maintenance intervals depend on the individual intensity of use. The prosthesis components must undergo a visual and functional inspection after the first 30 days of use.

## 12. Report on a serious incident

Any serious incident involving the product must be reported immediately to the manufacturer and the competent authorities, quoting the serial number. The serial number can be found either on the label on the packaging, underneath the pyramid adapter or in the front area on the underside of the prosthetic foot.

## 13. Disposal

The product and packaging must be disposed of in an environmentally friendly manner in accordance with local or national disposal regulations.

## 14. Liability

The manufacturer accepts liability provided that the product is used in accordance with the information and instructions in this document. The manufacturer is not liable for damage caused by failure to observe these instructions, in particular due to improper use or unauthorised modification of the product.

## 15. Conformity

This product has been tested in accordance with standard ISO 22675 with two million load cycles. The CE declaration of conformity can be downloaded from the manufacturer's website.

## 16. Name and address of the manufacturer

steptics GmbH  
Gmunder Str. 35  
81379 Munich, Germany  
[www.steptics.com](http://www.steptics.com)  
[support@steptics.com](mailto:support@steptics.com)

## 17. Technical data

Built height [mm]	280
Average weight [g]	645
Field of application	Recreational and mass sports
Material	Carbon fibre reinforced plastic

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